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Mindray Beneheart C1A / C2

Kontrolle Gerätestatus

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
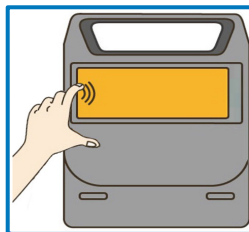
Abb. 3

Abb. 4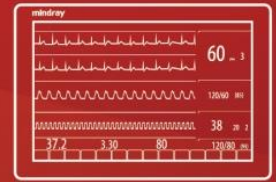

Country	Switzerland
Contact	Procamed AG
Project	Manual addendum
Device	BeneHeart C-Series

This memo is intended to explain chapter 8.2 of the BeneHeart C series manual.

Regarding "maintenance" under chapter 8.2. Mindray C-Series AED are maintenance free and do not require any special maintenance. However, the device needs 'inspection'. A visual inspection of the battery and pads. We recommend to do this inspection at least once per month.

8.2 Maintenance Safety Information

WARNING

- Failure for the responsible institution employing this equipment to implement a satisfactory maintenance schedule may cause undue equipment failure and delayed analysis.
- No modification of this equipment is allowed.
- This equipment contains no user serviceable parts.
- The safety checks or maintenance involving any disassembly of the equipment should be performed by professional servicing personnel. Otherwise, undue equipment failure and possible health hazards could result.
- The service personnel must be properly qualified and thoroughly familiar with the operation of the equipment.

The check and the replacement of the pads and battery can be replaced by anyone. No special training is needed. Pads and battery are not parts but disposables.

For any related questions, feel free to reach out to me.

Kind Regards,

Michel Rijnberk

Patient Monitoring Life Support
Business Development Manager AED - Europe

e-mail: michel.rijnberk@mindray.com

mobile: +31 645868895

G Inspection Record

Current Date: Month/Year: 2
Page: 1 of 1 in the corresponding file

Daily Checklist					
Inspected Date	Status Indicator Battery	Inspected by	Inspection Date	Status Indicator Pads	Inspected by
1.	<input type="checkbox"/> in green <input type="checkbox"/> in red		17.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
2.	<input type="checkbox"/> in green <input type="checkbox"/> in red		18.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
3.	<input type="checkbox"/> in green <input type="checkbox"/> in red		19.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
4.	<input type="checkbox"/> in green <input type="checkbox"/> in red		20.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
5.	<input type="checkbox"/> in green <input type="checkbox"/> in red		21.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
6.	<input type="checkbox"/> in green <input type="checkbox"/> in red		22.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
7.	<input type="checkbox"/> in green <input type="checkbox"/> in red		23.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
8.	<input type="checkbox"/> in green <input type="checkbox"/> in red		24.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
9.	<input type="checkbox"/> in green <input type="checkbox"/> in red		25.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
10.	<input type="checkbox"/> in green <input type="checkbox"/> in red		26.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
11.	<input type="checkbox"/> in green <input type="checkbox"/> in red		27.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
12.	<input type="checkbox"/> in green <input type="checkbox"/> in red		28.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
13.	<input type="checkbox"/> in green <input type="checkbox"/> in red		29.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
14.	<input type="checkbox"/> in green <input type="checkbox"/> in red		30.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
15.	<input type="checkbox"/> in green <input type="checkbox"/> in red		31.	<input type="checkbox"/> in green <input type="checkbox"/> in red	
Monthly Checklist					
Expiration date of electrode pads:					